- D** Blasenkatheterventil CareVent® mit Verbindungsschlauch
- GB** Bladder Catheter Valve CareVent® with Connection Tube
- BG** Клапа на катетър за пикочен мехур CareVent® с тръба за свързване
- CZ** Ventil močového katétru CareVent® se spojovací hadičkou
- DK** Blærekateterventil CareVent® med forbindelsesslange
- E** Válvula del catéter vesical CareVent® con tubo de conexión
- EST** Põiekatetri klapp CareVent® koos ühendustoruga
- F** Vanne de cathéter de drainage de vessie CareVent® avec tubulure de raccordement
- FIN** Virtsarakon CareVent®-katetri venttiilillä liitosletkun kanssa
- GR** Βαλβίδα καθετήρα ουροδόχου κύστης CareVent® με λάστιχο σύνδεσης
- H** CareVent® hólyagkatéter-szelep csatlakozósóvel
- I** Valvola per catetere vescicale CareVent® con raccordo
- LT** Šlapimo pūslės kateterio vožtuvas „CareVent®“ su sujungimo vamzdeliu
- LV** Urīnpūšļa katetra vārstulis CareVent® ar savienojšo caurulīti
- NL** Blaas katheterventiel CareVent® met verbindings slang
- P** Válvula de cateter vesical CareVent® com tubo de ligação
- PL** Zastawka do cewnika CareVent® z rurką łącznikową
- RO** Ventil de catetru de vezică Care Vent® cu furtun de legătură
- RUS** Клапан CareVent® для катетера мочевого пузыря с соединительной трубкой
- S** Blåskateterventil CareVent® med anslutningsrör
- SK** Ventil CareVent® k cievke do močového mechúra s prípojnou hadičkou
- SLO** Ventil CareVent® za urinski kateter in povezovalna cevka

GENERELLE INFORMATION

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INSTRUCTIONS FOR USE

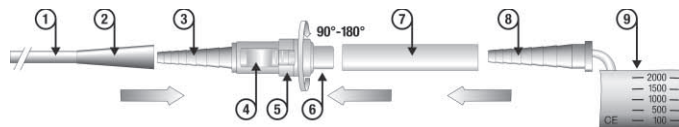
Bladder Catheter Valve CareVent® with Connection Tube

GENERAL INFORMATION

The product is provided single packed and sterile (EO sterilization). It is intended for single use only and may not be re-sterilized. The product must be stored in a cool, dry place and must be protected from daylight. Do not use this product if the packaging or the product is damaged.

DESCRIPTION

The bladder catheter valve CareVent® weighs only 6 gram and the following different components, as specified below, might be attached. It is intended to facilitate the drainage of the urine from the bladder to e.g. a urine bag or directly into the toilet.

Design

The product might be connected to only one or all of the following accessories, as stated in the drawing above and indicated within the context of this instruction of use. The drawing shall facilitate the use of the CareVent®.

- | | | |
|--------------------------------------|-----------------------------|--------------------------------------|
| ① Position to hold the catheter | ④ Lever | ⑧ Conical connector of the urine bag |
| ② Funnel shaped part of the catheter | ⑤ Locking ring | ⑨ Urine bag |
| ③ Conical connector of the valve | ⑥ Front end of the valve | |
| | ⑦ Connection tube (Silicon) | |

Note: Please refer to the label for details on the product specifications, etc.

Please Note: The above mentioned product/accessories must always be used in accordance with current medical standards regarding indications, contraindications and techniques. Please refer to your physician and or us for further details if the CareVent® is suitable for you and on the use of the CareVent®.

ADVANTAGES

- Reduced infection risk since no urine dropping
- Self-closing valve
- Easy disinfectable
- Easy to use, colour coded
- Light weight (only 6 gram)
- Suitable for all bladder catheter

INDICATION

Catheter bladder valve for external use only and for leakage controlled diversion of urine from the bladder to e.g. a urine bag.

CONTRAINDICATION

Inability of connecting the CareVent® to any accessories.

POSSIBLE COMPLICATIONS

- Sepsis/infections

PREPARATION

- Inspect packaging for damage
- Remove product from packaging
- Inspect product for damage

Note: Do not use the product if the packaging and/or the product are damaged.

DIRECTIONS FOR USE

Un-packaging

You can open the packaged product by simply holding both flaps of the packaging and tearing them apart without any additional tools. While doing so, please ensure, that the CareVent® and the connection tube do not drop on the floor and therefore getting contaminated.

Fixation of the CareVent® with the catheter

Insert the conical connector (3) firmly into the conical funnel of the catheter (2). Ensure that the lever (4) is pointing towards you, to facilitate later use.

Please Note: The design of the CareVent® guarantees the firm permanent connection to the catheter.

Emptying of the bladder according to the natural rhythm

To empty the bladder, please position the front end of the CareVent® (6) into the toilet or into a suitable container, big enough to accept all of your urine, and then pull the lever (4) back toward the catheter.

After emptying your bladder, please release the lever (4) and the valve will close automatically.

Use during the night

Connecting the CareVent® to an optional available urine bag (9) by using the included connection tube (7), might avoid the nightly toilet walk.

1. Insert (push) the front end of the valve (6) firmly into the included connection tube (7) and check for tight fitting.
2. Insert (push) the conical connector (8) into the included connection tube (7) and check for tight fitting.
3. Please verify that the connections are firmly connected.
4. Pull back the lever (4) towards the catheter and turn the locking ring (5) in either direction by one quarter to one half to secure the opened lever position and keeping the valve open to allow the urine to drain into the urine bag.
5. Please verify, that the tubes are not kinked, to avoid possible interruption of the urine flow.
6. In the morning, turn back the locking ring (5) into the original position allowing the lever (4) to return into the gap (original position). Please verify, that the lever (4) is moving forward towards the connection tube (7) and therefore closing the valve.
7. Disconnect the connection tube (7) from the front end of the valve (6) by simply withdrawing it from the valve to disconnect the urine bag.

Cleaning of the valve

The CareVent® has to be cleaned regularly. Remove the valve from the accessories as described below:

1. To avoid manipulation or an accidentally removal of the catheter, hold the catheter firmly at the "position to hold the catheter" (1) as indicated in the drawing and with your other hand hold the valve firmly.
2. Withdraw now the catheter from the valve by pulling firmly the valve and keeping the catheter in position.
3. Pull back the lever (4) towards the catheter and turn the locking ring (5) in either direction by one quarter to one half to secure the opened lever position and keeping the valve open.
4. Rinse now the open valve thoroughly with water and spray it with ready-to-use medical disinfectant.

Please Note: We recommend you to train the described steps and / or in case of questions to contact your physician or us, prior the first use of the easy to use CareVent®. We also recommend you prior the first use, to rinse the open valve thoroughly with water and spray it with ready-to-use disinfectant and purchase a sufficient amount of urine bags. Both can be obtained from us.

GENERAL PRECAUTIONS

- The packed devices have to be stored in a dry place at a temperature between 0°C and 30°C.
- The packed devices have to be protected from direct daylight.
- Do not use devices if the expiry date on the sterile packaging is exceeded.

DISPOSAL AFTER USE

After use medical devices, and accessories may be a potential biohazard. Handle and dispose them properly off according to approved and accepted medical practice and in compliance with the applicable and statutory local, state and federal laws and regulations.

Attention: Medithek GmbH products, and product kits are compatible, nevertheless the user must ensure prior the procedure the compatibility of the individual products for the specific application, especially if these are non Medithek GmbH products.

Guarantee

Medithek GmbH is only liable for guaranteeing that each individual product is manufactured, inspected and packed with the utmost care. THIS IS THE ONLY VALID GUARANTEE AND SUBSTITUTES ALL OTHER GUARANTEE DECLARATIONS WHICH HAVE BEEN GIVEN.

Since Medithek GmbH has no control/influence over indications, use and operation, it cannot accept be held responsible for effectiveness, complications or for the failure of the treatment, therefore Medithek GmbH assumes no liability for damages and costs. Products which have a defect for which Medithek GmbH is responsible, will be replaced.

Medithek GmbH individual products, product components and product kits are inter-compatible, nevertheless the user must ensure prior the procedure the compatibility of the individual products/components for the specific application, especially if these are non Medithek GmbH products.

Employees of Medithek GmbH are not authorised to deviate from the above terms and conditions, to accept any extended liability or additional product-related responsibilities.

Products are subject to alterations without prior notice.

DESCRIPTION OF SYMBOLS

 Single use only	LOT Batch number	REF Order code	 Shelf life
 Manufacturing date	 Refer to instruction of use	 Protect from direct sunlight	
 Store in a dry place	 Store in a cool place	 Sterilized by ethylene oxide	

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medithek GmbH
Medithek GmbH, Willinghusener Weg 5a, 22113 Oststeinbek, Germany